



EZ-SCREEN[®] Cup

Quick Reference Guide

Drug tests for *in vitro* diagnostic use only
 Adulterant tests are for forensic use only

The device detects one or more of the following drug classes at or above the following cutoff concentrations:

Amphetamine (AM)	300 ng/ml	Methadone (MT)	200 ng/ml
Barbiturates (BA)	200 ng/ml	Opiates (OP)	100 ng/ml
Benzodiazepines (BZ)	200 ng/ml	Oxycodone (OX)	100 ng/ml
Buprenorphine (BU)	10 ng/ml	Phencyclidine (PC)	25 ng/ml
Cocaine (CO)	100 ng/ml	Propoxyphene (PP)	300 ng/ml
Methamphetamine (mA)	1000 ng/ml	Cannabinoids (TH)	40 ng/ml

MEDTOX Diagnostics, Inc. • 1238 Anthony Road
 Burlington, NC 27215
 Phone: 336-226-6311 • Fax: 336-227-7302
 E-mail: sales@medtoxdiagnostics.com

Instructions

<p>1. Open pouch and remove device. Note: product must be allowed to come to room temperature (18-25°C / 64-77°F) if it is stored cooler than that.</p> <ul style="list-style-type: none"> •Acceptable Storage Temp: (2-25°C / 36-77°F) •Do not use expired product. 	<p>2. After opening pouch, remove lid and DISCARD desiccant.</p>	<p>3. Donor voids into cup. The Urine volume should be at or above the Fill Line.</p> <ul style="list-style-type: none"> •Always keep container upright and minimize handling during test development. •TIGHTEN lid securely onto the cup to avoid potential loss/leaking of sample if shipment to lab is required. 	<p>4. For panels with Adulterant Strip only -- Turn the cup and read the Adulterant strip by comparing it to the color chart provided.</p> <ul style="list-style-type: none"> •Tests begin developing upon voiding into the cup. •A timing device (like a wall clock, kitchen timer, watch or cell phone) is recommended.

READING THE TEST RESULTS:		INTERPRETING RESULTS:		VALID: A reddish-purple line appears in the "C" area for each test strip.	
<p>5. Remove privacy tab to read results.</p> <p>Read Time: 5 minutes after voiding into cup</p>	<p>•If needed, tilt cup back slightly so you can see lines clearly.</p> <p>NOTE: Line intensity may vary. The presence of any reddish-purple line, whether faint or broken, is to be interpreted as a negative result.</p>	<p>☆ INVALID: Above Test is Invalid-- No control line "C" present. Retest donor with a new device if you get an invalid result.</p>	<p>NEGATIVE: Any reddish-purple line (even a faint or broken line) visible at 5 minutes in the test area (along with a valid control) is a negative result.</p> <p>Example above: all five test results are negative, and test is Valid.</p>	<p>PRESUMPTIVE POSITIVE: The absence of a line in the test area (along with a valid control) is a presumptive positive result.</p> <p>Example above: the OP (Opiate) test is presumptive positive and the test is Valid.</p>	

For complete product information refer to the EZ-SCREEN[®] CUP Package Insert-I, Rev. 8/11 available at <http://www.medtox.com/ProductTraining.aspx> or contact Technical Support at 1-877-643-5703.